

VITA AKZENT® Plus

Product information



VITA shade taking

VITA shade communication

VITA shade reproduction

VITA shade control

Date of issue: 03.13

VITA shade, VITA made.

VITA

Optimizes everything. Effortlessly.
Stain glazing, masking and glazing – regardless of the coefficient of thermal expansion (CTE)

VITA AKZENT® Plus Optimizes everything. Effortlessly.



Greater freedom for your practice and laboratory

Are you looking for a simple yet effective approach to stain glazing, coloring and glazing? Then VITA AKZENT Plus is the product line for you.

This system comprising stain glazing, masking and glazing shades guarantees unbeatable freedom for your practice and laboratory.

It is suitable for all dental ceramic materials and any CTE.

New shades, new applications, new indications – with VITA AKZENT Plus the choice is yours.

Be it powder, paste or spray, you decide how you want to use VITA AKZENT Plus for maximum cost-efficiency that suits your needs.

VITA AKZENT® Plus SPRAY Even application at the touch of a button



AKZENT Plus SPRAY KIT		
Product line content		
Product	Shade	Content
BODY SPRAY	BS01 – BS05	5 x 75ml
GLAZE SPRAY		1 x 75ml
Accessories		



Particularly suited to users in a practice or laboratory environment who, for example, wish to adapt the shade of monolithic restorations.

The spray glaze and spray finishing agent shades are ready to use and easy to apply.

Benefits:

- Even application for easy, uniform surface treatment
- Precise spray application without any wastage as a result of scatter thanks to a special new spray head



VITA AKZENT® Plus POWDER Combining flexibility and individuality with cost-effectiveness



AKZENT Plus POWDER KIT		
Product line content		
Product	Shade	Content
EFFECT STAINS	ES01 – ES14	14 x 3g
BODY STAINS	BS01 – BS05	5 x 3g
GLAZE		1 x 5g
GLAZE LT		1 x 5g
FINISHING AGENT		1 x 5g
POWDER FLUID		1 x 20ml
Accessories		



The VITA AKZENT Plus POWDER materials are ideal for dental technicians who wish to use them for characterizing conventional veneering ceramics as well as the fine-structure feldspar ceramic VITA PM 9.

Benefits:

- Unlimited flexibility and cost-effectiveness
- Can be mixed freely with one another to obtain further custom shade effects



VITA AKZENT® Plus PASTE Naturally vivid shades



AKZENT Plus PASTE KIT		
Product line content		
Product	Shade	Content
EFFECT STAINS PASTE	ES01 – ES14	14 x 4g
BODY STAINS PASTE	BS01 – BS05	5 x 4g
GLAZE PASTE		1 x 4g
GLAZE LT PASTE		1 x 4g
FINISHING AGENT PASTE		1 x 4g
PASTE FLUID		1 x 15ml
Accessories		



VITA AKZENT Plus PASTE stains are perfect for fast and simple characterization of restorations made from VITABLOCKS, for example.

Benefits:

- Ready-to-use paste with uniform consistency and homogeneous pigmentation for fast application
- Can be mixed with one another to obtain custom shade effects
- Can be diluted or mixed again




VITA AKZENT Plus PASTE FLUID provides ideal consistency if the pastes are dried out.

VITA AKZENT® Plus Naturally vivid shades



AKZENT Plus shade selection						
Product line						
Product	Sample shades	Shade	Prod. ID	Available as		
				Spray	Powder	Paste
BODY STAINS – Translucent glazing stains – For modifying the shade effect of the base material		Yellow	BS01			
		Yellow-brown	BS02			
		Orange	BS03			
		Olive grey	BS04			
		Grey-brown	BS05			
EFFECT STAINS – Masking stains that offer superior coverage – For natural surface effects – For coloring during layering		White	ES01			
		Cream	ES02			
		Lemon yellow	ES03			
		Sunshine yellow	ES04			
		Orange	ES05			
		Russet	ES06			
		Khaki	ES07			
		Pink	ES08			
		Dark red	ES09			
		Lilac	ES10			
		Blue	ES11			
		Grey-blue	ES12			
		Grey	ES13			
		Black	ES14			
GLAZE – For uniform surfaces that shine						
GLAZE-LT – Low-melting glazing agent for ceramics with a firing temperature below 800°C						
FINISHING AGENT – For silk-matte surfaces						

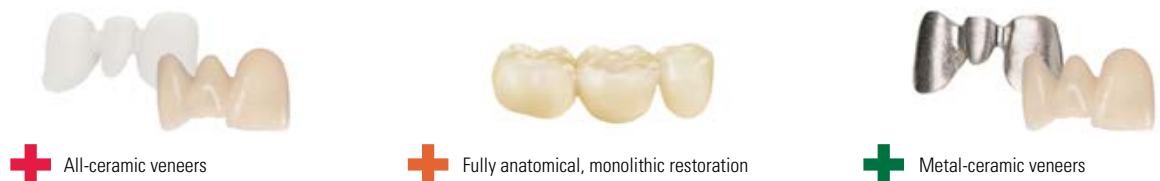

+ All-ceramic veneers

+ Fully anatomical, monolithic restoration

+ Metal-ceramic veneers



+ Fine-structure feldspar ceramic blocks



+ Press ceramics

+ For extra versatility

Stain glazing

System now includes: glazing stains (Body Stains).
The translucent stains provide an effect similar to that of a thin shade filter.
They are thus ideally suited to:

- Surface staining
- Modifying the shade effect of the base material

Masking

Extremely fine-grained: masking stains (Effect Stains).
These stains are primarily distinguished by:

- intensive shade effects and
- significant coverage

They are therefore particularly suited to the application of natural surface effects and internal mixing of colors.

Glazing


For a finish that shines:

- VITA AKZENT Plus GLAZE – the glazing agent for a uniform surface that shines like glass
- VITA AKZENT Plus GLAZE LT – the low-melting glazing agent for ceramics with a firing temperature below 800 °C
- VITA AKZENT Plus FINISHING AGENT – the finishing agent for silk-matte veneer surfaces

+ For extra processing convenience

All stains are fluorescent, exceptionally fine-grained and offer excellent shade stability.
The thixotropic consistency of the powders and pastes also facilitates precise application of the material in question.

With the unique VITA SYSTEM 3D-MASTER, all natural tooth shades can be systematically determined and perfectly reproduced.

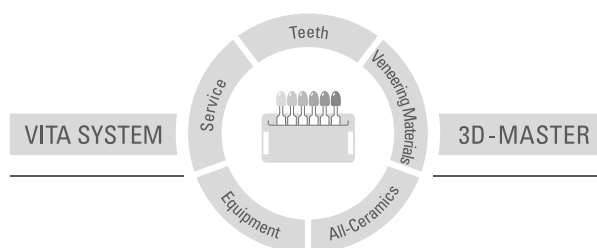

Please note: Our products must be used in accordance with the instructions for use. We accept no liability for any damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of application. We cannot accept any liability if the product is used in conjunction with materials and equipment from other manufacturers that are not compatible or not authorized for use with our product. Furthermore, our liability for the accuracy of this information is independent of the legal basis, and in as far as legally permissible, shall always be limited to the value as invoiced of the goods supplied, excluding value-added tax. In particular, as far as legally permissible, we do not assume any liability for loss of earnings, indirect damages, ensuing damages or for third-party claims against the purchaser. Claims for damages based on fault liability (culpa incontractando, breach of contract, unlawful acts, etc.) can only be made in the case of intent or gross negligence. Date of issue of these instructions for use: 03.13

VITA Zahnfabrik has been certified in accordance with the Medical Device Directive and the following products bear the CE mark  0124:

VITA AKZENT® Plus

US 5498157 A · AU 659964 B2 · EP 0591958 B1

VITA

VITA Zahnfabrik H. Rauter GmbH & Co. KG
Postfach 1338 · D-79704 Bad Säckingen · Germany
Tel. +49(0)7761/562-0 · Fax +49(0)7761/562-299
Hotline: Tel. +49(0)7761/562-222 · Fax +49(0)7761/562-446
www.vita-zahnfabrik.com · info@vita-zahnfabrik.com
 facebook.com/vita.zahnfabrik